

CLIPSULIN - English - Welcome guide



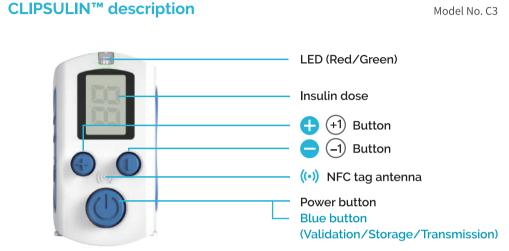

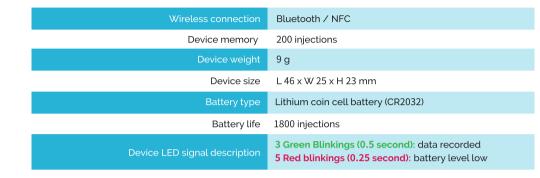

If you have any questions about the use of the CLIPSULIN™, do not hesitate to visit our website www.diabnext.com or to contact us by email: support@diabnext.com Our team is here to help you. (€ FC 🔊 (((CCAN18LP0200T0 € 005-101731 CM||T |D: 2018DJ1502 FCC |D: 2AOKVC\$03



Set up

1 Download DIABNEXT™ APP











Install the battery in the device. Turn on the Bluetooth of your smartphone. To pair your device with the mobile app, open your DIABNEXT™ app, then press the **CENTRAL BLUE BUTTON** on your CLIPSULIN™. Then, follow the instructions in your DIABNEXT™ app.




Automatic synchronization of the CLIPSULIN™ with the app

After pairing your device, the data will be automatically transmitted if you keep your smartphone's Bluetooth turned on and the DIABNEXT™ app running in the background*.

*Even if not open, an app continues to run in the background of your smartphone. If you quit the app on the other hand, it won't receive the data automatically.

Using the CLIPSULIN™

2 Attach CLIPSULIN™ to your insulin pen. The CLIPSULIN™ display screen should be on the side of the dose windows display of your insulin pen. Always prime your pen **BEFORE** turning on the CLIPSULIN™.

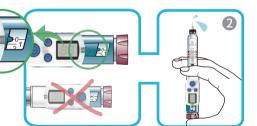

Only if necessary, use 4 😛 or 😑 to adjust the dose on the screen.



When you validate the dose on your CLIPSULIN™, 6 the data is transferred automatically to your logbook*.

*It is not necessary to re-open the DIABNEXT $^{\text{\tiny{TM}}}$ app to automatically transfer your data. If you have not quit it, your app is still running in the background and can save the data transmitted by Bluetooth.



Enjoy your digital logbook & communities.

When the dose shown on the CLIPSULIN™ display screen matches the dose dialed in your insulin pen, confirm by pressing the CLIPSULIN™ CENTRAL BLUE BUTTON*.

*Pressing the blue button does not turn off your device. Your CLIP-







Turn on the Bluetooth of your smartphone. Press the CLIPSULIN™ button (CENTRAL BLUE **BUTTON**) and then dial on your pen the dose of insulin you want to inject. 3